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The following table provides a definition of the signal words used in the instructions for use in connection with the safety instructions.

### Safety instructions

#### WARNING!

Do not place the leg bag straps twisted, too loose and/or too tight. This can cause constrictions, circulatory problems and even blood pooling. There is a risk of thrombosis.

All rights reserved. No part of these instructions for use may be reproduced in any way or processed, duplicated or otherwise distributed using electronic systems without the express written consent of the manufacturer.

#### CAUTION!

Due to friction between the skin and the leg bag straps, reddening of the skin and skin injuries may occur.

Application duration depends on how long each leg bag is worn for. Attention should be paid to skin changes and blood circulation.

The product is a single-patient product and is therefore intended exclusively for use – including multiple use – on a single patient.

### Product description

#### Working principle

The leg bag is first fixed in place using the leg bag straps. Due to the continuous Velcro surface, the leg bag can be worn on the upper or lower leg and mobility for the patient can be achieved.

The leg bag straps are suitable for leg bags with and without a drip chamber.

#### Application

Common urine leg bags, especially sanabelle leg bags, can be fastened securely and comfortably to the leg with sanabelle leg bag straps U900:

- Take the leg bag and the leg bag straps out of the packaging.
- Take hold at the top of the edge of the strap with the two Velcro strips and the white Velcro strip. Pass the green Velcro strips of that strap through the pre-punched openings at the top edge of the leg bag. (Image ①)

#### Explanation of the warnings included in these instructions for use

The following table provides a definition of the signal words used in the instructions for use in connection with the safety instructions.

### Explanation of the warnings included in these instructions for use

#### Intended use

##### Medical purpose

Fastening of a leg bag to the upper or lower leg.

##### Indication

##### Urinary incontinence.

##### Contraindications / Limitations

In patients with circulatory disorders, phlebitis or diabetes mellitus, attention must be paid regarding impairment of venous blood flow.

##### Intended target patient group(s)

For the care of mobile individuals with moderate to severe urinary incontinence where urine is collected via internal or external drains.

##### Intended users

Country-specific differences in instruction and training must be taken into account.

Patient and relatives of the patient: The product is intended for home use by the patient himself/herself or relatives of the patient. Patients and relatives of the patient may only use the product after receiving product-specific training.

##### Disposal

When disposing of the packaging and components, the applicable national disposal regulations in your country must be observed.

##### Legal notice

#### WARNING!

Improper handling and improper use can cause hazards and damage.

med1one medical gmbh assumes no liability for damage caused by unauthorised product modifications or improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use.

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