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sanabelle manguito para bolsa de pierna U910

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GB Instructions for use

sanabelle leg bag sleeve U910

The sanabelle leg bag sleeve U910 keeps the leg bag fixed securely to the leg. The whole surface of the urine bag is pressed against the leg and this prevents the bag from moving around and helps to dampen any undesired sounds.

The elasticated fabric of the leg bag sleeve prevents uncomfortable constrictions or pressure sores and is very comfortable to wear.

- Can be worn on upper and lower leg
- For leg bags with and without drip chamber
- Suitable for leg bags carrying up to 750 ml volume
- Material: 63 % polyester, 37 % elastane
- Latex-free

Directions for Use:

(Image left as an example size M/blue)

- ① Strap the end of the leg bag sleeve over the leg with the coloured size marking (yellow, blue, brown or green) at the top and with the seam showing on the outside. The leg bag sleeve must lie flush against the leg, with no folds or creases.
- ② Insert the drainage valve of the leg bag into the valve opening of the leg bag sleeve U910 (marked in red).
- ③ Pull the bottom end of the sleeve upwards over the leg bag.
- ④ Align the bag sideways for a comfortable fit.

Size guide circumference upper/lower leg

Size	cm	inches
S	25–33	10–13
M	40–50	15½–19½
L	50–60	19½–23½
XL	60–70	23½–27½

Precautions

Double check that you have selected the correct size. Carrying a leg bag sleeve that is too narrow can lead to constrictions, circulatory disorders and even blood congestion. There is a risk of thrombosis.

Attention

Skin irritation and abrasions may result from friction between the skin and the leg bag sleeve. In patients with circulatory disorders, inflammation of the veins (phlebitis) or diabetes mellitus, the risk of restricting venous blood flow should be taken into account.

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