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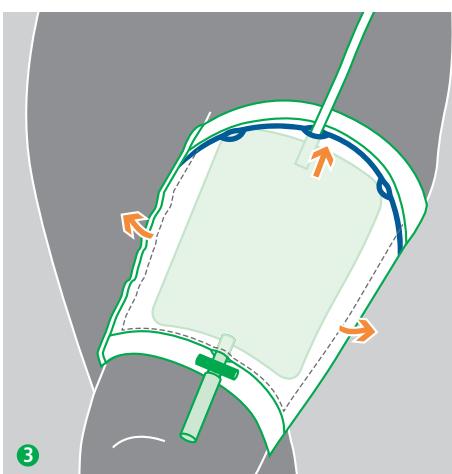
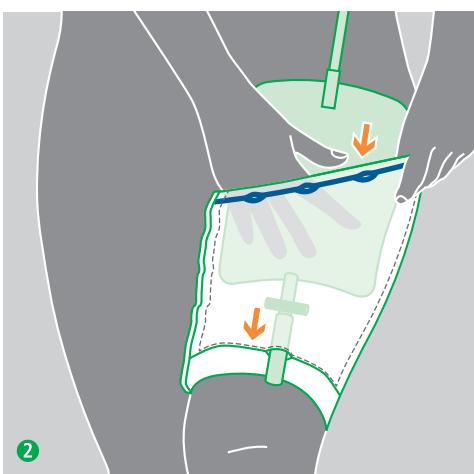
## GB Instructions for use

### sanabelle leg bag pocket U920

The sanabelle leg bag pocket U920 keeps the leg bag fixed securely to the leg. The whole surface of the urine bag is pressed against the leg and this prevents the bag from moving around and helps dampen any undesired noises. The elasticated fabric of the leg bag holder prevents uncomfortable constrictions and is very comfortable to wear.

- Can be worn on upper and lower leg
- For leg bag with and without drip chamber
- Suitable for leg bags carrying up to 750 ml volume
- Material: 80 % polyester, 20 % elastane
- Latex-free

#### Directions for Use:

(Image left as an example size M/blue)

- ① Strap the sanabelle leg bag pocket U920 over the leg with the coloured size marking (yellow, blue, brown or green) at the top and the seam showing on the outside. The leg bag pocket must lie flush against the leg, with no folds or creases.
- ② Place the leg bag in the pocket. Insert the drainage valve of the leg bag into the valve opening of the leg bag pocket.
- ③ Align the bag sideways for a comfortable fit. For additional hold, the inlet tube can be fed through one of the three fixing holes, as long as the tube is not connected to a draining incontinence product.

#### Size chart circumference upper/lower leg

Size	cm	inches
S	30 – 45	12 – 17½
M	45 – 65	17½ – 25½
L	55 – 80	21½ – 31½
XL	65 – 90	25½ – 35½

#### Precautions

Double check that you have selected the correct size. Carrying a leg bag pocket that is too narrow can lead to constrictions, circulatory disorders and even blood congestion. There is a risk of thrombosis.

#### Attention

Skin irritation and abrasions may result from friction between the skin and the leg bag pocket. In patients with circulatory disorders, inflammation of the veins (phlebitis) or diabetes mellitus, the risk of restricting venous blood flow should be taken into account.

#### Washing instructions

- |                   |               |                   |             |                  |
|-------------------|---------------|-------------------|-------------|------------------|
|                   |               |                   |             |                  |
| Washable at 60 °C | Do not bleach | Dry at max. 60 °C | Do not iron | Do not dry clean |

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