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### 2. Product description

#### 2.1. General

Sanabelle HME-A6 comprises a plastic housing with an insert of hydroscopic foam material. The HME has a 15 mm female connector, which can be fitted to the standardised 15 mm connector of an endotracheal tube or a tracheostomy tube, and is equipped with a 5-mm port for the supply of oxygen.

#### 9. Reinigung/ Desinfektion

These HMEs must not be used a second time and must also not be cleaned or disinfected.

The safety valve minimizes the risk of occlusion of the humidifying medium, allows short-term high tidal volume and coughing up of larger quantities of secretions.

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### GB sanabelle HME-A6

Artificial nose with safety valve and oxygen connector

#### General Information

Please read the instructions for use through carefully each time before you use the products. The

products must only be used by doctors, individuals instructed in their proper use and individuals entrusted with patient care.

- For use on a single patient only!

### 1. Intended use

Sanabelle HME-A6 (artificial nose/ heat and moisture exchanger according to EN ISO 9360-2) is intended for the respiratory gas humidification of inspiration air.

### 2. Product description

#### 2.1. General

Artificial noses/ Heat and moisture exchangers (HMEs)

- extract the moisture and heat from the patient's expiration air and return them to the patient via the inspiration air;
- prevent the accumulation of tough secretions in the trachea and reduce infections;
- filter the respiratory air and prevent particles from entering the patient's airways.

#### 9. Cleaning/ Disinfection

These HMEs must not be used a second time and must also not be cleaned or disinfected.

#### 10. Storage/ Warehousing

Sanabelle HMEs-A6 which have not yet been used should be stored in a dry environment, protected from direct sunlight and/or heat/extreme cold.

#### 11. Disposal

The devices must be disposed of in accordance with the respectively applicable national regulations governing waste materials.

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### 3. Period of use

Replace the sanabelle HME-A6 after a maximum of 24 hours to counteract any possible bacterial or fungal growth. The HME must be replaced more frequently depending on the needs, for example, if the breathing resistance is in danger of increasing due to accumulations of secretion.

### 4. Indications

For the efficient filtering, warming and humidifying of inhaled air for patients with spontaneous breathing through an endotracheal tube or tracheal cannula, in hospital or at home.

In case of high levels of secretions, the heat and moisture exchanger (HME) will need to be replaced with a new one more regularly.

### 5. Contraindications

- The sanabelle HME-A6 must not be used outside the recommended tidal volume range, in order to prevent an increased concentration of CO<sub>2</sub> in case of a low tidal volume, or insufficient humidification, if the tidal volume is too high.

Please observe the doctor's recommendation. Please consult a specialist in case of questions.

- The device is not to be used on patients with strong exsiccosis or excessive secretions in the lungs and airways or in case of leaking gas flow (expired gas does not flow through the humidifier).

These HMEs must not be used a second time and must also not be cleaned or disinfected.

#### 10. Storage/ Warehousing

Sanabelle HMEs-A6 which have not yet been used should be stored in a dry environment, protected from direct sunlight and/or heat/extreme cold.

#### 11. Disposal

The devices must be disposed of in accordance with the respectively applicable national regulations governing waste materials.

#### 9. Reinigung/ Desinfektion

Der sanabelle HME-A6 is for single use only! Cleaning, disinfecting or re-sterilising it for reuse may impair the function and increase the risk of transmitting infections.

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Sanabelle HME-A6 is latex-free, DEHP-free and electrically non-conductive.

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There are no known side effects

### 7. Warnings/ Safety precautions

1. Always ensure the patency of the HME.
2. Monitor patients for potential fatigue!

Upon indication of difficulty breathing, remove the HME immediately! Reason: the HME's resistance requires increased breathing effort.

4. The sanabelle HME-A6 is for single use only!

Cleaning, disinfecting or re-sterilising it for reuse may impair the function and increase the risk of transmitting infections.

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