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GB sanabelle U510 intermittent catheter PRO hydrophilic coated

Indication

For intermittent single catheterisation of the bladder. For taking diagnostic samples or for therapeutic purposes.

Description

The sanabelle U510 intermittent catheter consists of DEHP-free PVC and is latex-free. The coating contains polyvinylpyrrolidone (PVP). The integrated water sachets filled with sterile water, are used to activate the hydrophilic coating of the catheter.

Contraindications

- Do not use in case of known intolerance to PVC or PVP.
- Before performing intermittent catheterisation for the first time, a medical examination should be carried out in order to rule out contraindications.
- Contraindications are injuries to the urethra and the bladder (e.g. urethral constriction, urethral rupture, urethral bleeding, acute urethral inflammation)

Application

To empty the bladder, the catheter is inserted into the urethra by the user or by medical specialists.

- 1 Thoroughly wash hands with soap and water and disinfect them. Take care to avoid touching the catheter shaft directly with your hands during catheterisation.
- 2 Hold the catheter packaging vertically. Kink and compress the water sachet so that the pressure is directed downwards.
- 3 The water sachet opens and the sterile water enters the catheter packaging. Take care that the water is evenly distributed. If necessary, distribute the water through the packaging or carefully swing it to and fro. This enables complete activation of the hydrophilic coating to enable the catheter surface lubricious. Wait for at least 30 seconds.
- 4 The catheter packaging can be hung by the adhesive label at the rear to smooth and dry area.
- 5 Clean the area around the urethral orifice.
- 6 Catheterisation must be carried out under aseptic conditions. Open the catheter packaging and peel it apart as far as the lower edge of the catheter funnel or the protective sleeve. The tab makes it easy to open the packaging even if you have difficulty using your hands.
- 7 Slowly pull the catheter out of the packaging by the catheter funnel (7a). For catheters with a protective sleeve, the protective sleeve can also be used for sterile draw of the catheter (7b).
- 8 Carefully insert the catheter into the urethra until urine runs out (8a). For catheters with a protective sleeve, the catheter can be inserted by sliding the protective sleeve to avoid direct contact of the hands with the catheter. Slowly pull out the catheter when no further urine flows (8b).
- 9 Dispose of the used catheter and the packaging. Wash hands thoroughly after catheterisation.

Information for use

- The size of the catheter is selected individually for each patient according to medical standards. Please consult a specialist in case of questions in order to prevent damage to the urethra.
- Intermittent catheterisation by inexperienced persons may only be carried out under the instruction of the doctor or qualified medical specialists.
- The catheter and the integrated water sachet must not be used if any damage to the product is apparent.
- Do not use if the sterile packaging is damaged. Sterility is only ensured if the single packaging is undamaged.
- Use the catheter immediately after removing the activated catheter from the packaging.
- The catheter is only intended for transient use. Use over a period of several hours is not permissible.
- Re-use may result in cross-contamination. The catheter must not be cleaned, disinfected or re-sterilised.

- If the catheter comes into contact with a non-sterile object, dispose of it and use a new, sterile intermittent catheter.

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