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GB Instructions for use

sanabelle oxygen connecting tube for sanabelle HME-A6 and HME-A7

Indication

For the additional administration of oxygen when using the sanabelle HME-A6 and HME-A7 for patients requiring an external oxygen supply. The oxygen is warmed and humidified by the HME. It is 2. Securely connect the oxygen connecting tube to the external oxygen source.

3. Ensure an oxygen flow can be felt at the other end of the tube.

4. Connect the tube to the HME.

IMPORTANT: If the functionality of the external oxygen source and its connection to the oxygen tube is ensured, and yet no oxygen flow can be felt at the end of the tube, remove the oxygen connecting tube and replace it by a new one.

Usage

1. Prior to every use, make sure the external oxygen source is functioning correctly. Observe the instructions for use of the oxygen source used.

and HME-A7 on a patient. With that, a cycle of application may mean several replacements of the HME. In the event of dirt or accumulation of condensate in the tube, replace immediately.

- The oxygen connecting tube does not contain a device to implement safety of the patient to reduce the probability of fire. The end user is responsible for using a device corresponding to the definition in the EN ISO 8359/A1 standard. The corresponding device is available from medi1one.

- Always ensure the patency of the HME (see instructions for use of the sanabelle HME-A6 and HME-A7).

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